

# VIRTUAL FIELD VF-3

**LOMBART**  
ADVANCING EYECARE™

## ADVANCED TESTING & ANALYSIS SOFTWARE

The Virtual Field is the visual field test that your patients will love. Advanced testing and analysis software includes all the features you expect in one easy-to-use interface.



### PROGRESSION ANALYSIS

Track visual field loss over time with our advanced progression analysis software.

### MONOCULAR

Stimuli in the virtual reality headset are presented to each eye independently.

### GLOBAL ACCESS

Access results from any laptop, tablet, or mobile device with our HIPAA compliant website.

### INTEGRATED

Reports can be downloaded as PDF files and uploaded into any EHR.





## SAVES TIME

- Reduced testing time
- Test multiple patients at once
- Audio instructions free up your technicians
- Easier to clean than traditional perimeters
- No eye patch necessary



## SAVES MONEY

- Qualifies for ADA tax credit
- More affordable than traditional perimeters
- Lower maintenance costs



## INCREASED ACCESSIBILITY

- Mobile and lightweight design
- Ability to test anywhere with wifi
- More manageable for wheelchair patients.

## GOT QUESTIONS?

The questions below are frequently asked by doctors who are new to our platform. If your question is missing, please contact us and we'll be happy to help.

### CAN I BILL INSURANCE?

Yes, Virtual Field can bill codes 92081 (Superior 36), 92082 (screening) and 92083 (BOLT, full threshold). Test results are always available in your account.

### IS YOUR DEVICE SUPPORTED BY RESEARCH?

Yes, several studies showing correlation between Virtual Field and Humphrey Field Analyzer have been completed and several others are in progress.

### IS AN EYE PATCH REQUIRED?

No, stimuli are only presented to one eye at a time in the headset, so no eye patch is required and the test is not paused to switch eyes.

### WHAT IS YOUR REGULATORY STATUS?

Virtual Field is a Class I medical device and is registered and listed with the FDA. We follow all applicable regulatory requirements.

### HOW DO I CORRECT FOR REFRACTIVE ERROR?

Two trial lens adapters are included in each headset for vision correction using spherical equivalents.

### HOW IS THE DEVICE CLEANED?

Each headset comes with 100 sanitary pads that stick onto the headset.

We also recommend cleaning with alcohol wipes between each use.

### WHAT IS THE BOLT STRATEGY?

BOLT is our proprietary thresholding strategy that decreases the total testing time by 2-4 minutes with industry standard threshold prediction methods.

### WHAT LANGUAGES ARE AVAILABLE?

Our system administers instructions to, encourages, and monitors the patient in 34 languages. Contact us if you need a language that is missing.

### WHAT IS YOUR WARRANTY POLICY?

Your device is covered by a 2-year warranty.

### IS YOUR DEVICE AVAILABLE IN MY COUNTRY?

While we only serve US doctors at this time, we are working on regulatory approvals for other countries. Please contact us directly.

### DO YOU OFFER PROGRESSION ANALYSIS?

Yes, progression analysis reports can be viewed for patients with two or more exams. The reports can use one or two tests as a baseline.

### HOW DO I VIEW TEST RESULTS?

Test results can be viewed as a PDF in your account. From there, you can download the file and upload it to your EMR or print it out.

### IS IT COMFORTABLE?

Yes, taking a Virtual Field exam is considerably more comfortably than taking a conventional perimetry test.

### CAN MY WHEELCHAIR PATIENTS USE IT?

Yes, unlike conventional perimeters, elderly and wheelchair patients will not find it physically difficult or impossible to take a virtual exam.

### IS THERE EYE TRACKING?

No, our current model does not have eye tracking capabilities. We constantly consider and test new technology but only offer what works reliably.

### DO I HAVE TO SIGN A BAA?

Yes, all of our customers sign a Business Associate Agreement (BAA) when they place an order.

### HOW LONG DOES THE BATTERY LAST?

The headset battery should last 8 to 10 BOLT exams. We include a 10 foot charging cable in case you need to charge during a test.

### WHAT METRICS DO YOU DISPLAY?

Visual Field Index (VFI), Mean Deviation (MD), Pattern Standard Deviation (PSD), and Glaucoma Hemifield Test (GHT).

CONTACT LOMBART AT **800-LOMBART**